



Nebulizer Kit

Intended use:

The nebulizer kit is a device placed over a patient's nose or mouth to provide aerosolized humidification or medication for patients to inhale.

Instruction for Use:

1. Place the spray mask over the patient's face and place the elastic band behind the head to adjust the proper tightness.
2. Open the top of the nebulizer upper cover, inject the medicine according to the doctor's instructions, and then install the upper cover. When operation, the nebulizer needs to be used vertically, open the gas supply source switch, set the gas flow rate 6~8 (LPM), that is, nebulizer; , flow setting and use time, please follow the doctor's instructions and connect with the nebulizer mask to achieve therapeutic effect.

Warning :


1. The product is for single patient use, the maximum recommended is 7 days.
2. Please confirm the integrity of all parts and packaging before use. If there is any doubt or damage, please do not use this product.
3. This instruction is only a general indication, and it still needs to be used by professional medical personnel.
4. Dispose of in accordance with the local medical waste disposal regulations.
5. The device is for single use only. Any modification, rework, or sterilization will result in device malfunction or failure; and will also increase the risk of cross-infection of disease.
6. **Discontinue** use if discomfort occurs, and consult your physician.

Specification : Child/Adult

Manufacturing date: 2018/07/23

Expiratory date: 2021/07/22

Manufacturing lot: 180723

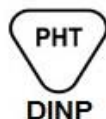

Obelis s.a. Boulevard Général Wahis 53 1030 Brussels, BELGIUM
Tel : + (32) 2.732.59.54 Fax : + (32) 2.732.60.03
E-Mail : mail@obelis.net



DA CHUNG MEDICAL CO., LTD
No. 600, Sec. 1, Dong Shan Rd., Dong Shan Township, Yilan County, Taiwan.
<http://www.dc-med.com.tw> Tel: 886-3-9594777