



Nebulizer Handheld Kit

Intended use:

The Nebulizer handheld kit is a device to provide aerosolized humidification or medication for patients to inhale.

Instruction for Use:

1. Open the cover of the small volume nebulizer and inject the medicine as directed by your doctor and then install the cover.
2. Connect the left and right connectors of the T connector to the mouthpiece and the short tube, and then connect the bottom to the nebulizer.
3. The oxygen tube is connected to the nebulizer and the flow meter.
4. When operation, nebulizer should be used vertically. Turn on the gas supply switch and set the gas flow rate of 6~8 (LPM), which means spray, dosage, flow setting and use time. Please follow the doctor's instructions.

Warning :


1. The product is for single patient use, the maximum recommended is 7 days.
2. Please confirm the integrity of all parts and packaging before use. If there is any doubt or damage, please do not use this product.
3. This instruction is only a general indication, and it still needs to be used by professional medical personnel.
4. Dispose of in accordance with the local medical waste disposal regulations.
5. The device is for single use only. Any modification, rework, or sterilization will result in device malfunction or failure; and will also increase the risk of cross-infection of disease.
6. **Discontinue** use if discomfort occurs, and consult your physician.

Specification : 18mmOD small nebulizer set

Manufacturing date: 2018/07/23

Expiratory date: 2021/07/22

Manufacturing lot: 180723

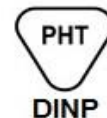

Obelis s.a. Boulevard Général Wahis 53 1030 Brussels, BELGIUM
Tel : + (32) 2.732.59.54 Fax : + (32) 2.732.60.03
E-Mail : mail@obelis.net



DA CHUNG MEDICAL CO., LTD
No. 600, Sec. 1, Dong Shan Rd., Dong Shan Township, Yilan County, Taiwan.
<http://www.dc-med.com.tw> Tel: 886-3-9594777